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65462 Ginsheim-Gustavsburg

Panty: 100% Baumwolle

Saugeinlage: 78% Polyester | 17% Baumwolle | 5% Polyamid

REF 0416.1

GB | Incontinence pants for men

1 | Delivery

1 x Incontinence pants

Read all of these instructions carefully! 

2 | Intended purpose

To alleviate the effects of mild incontinence by absorbing uncontrolled urine output.

- Incontinence pants specially developed for men.
- Absorbent pad is incorporated directly into the briefs, absorbs and processes moisture:
- Surface dries in a short time = low rewetting, therefore comfortable on the skin

For private use

3 | Contraindications / Not suitable

In the following conditions, wearing the incontinence pants is recommended only after consultation with your doctor:

- Skin problems/injuries to the part of the body where the support is worn, especially if there are signs of inflammation, swelling, redness and overheating
- allergic reactions

4 | Notes on safe use of the product

To date, there are no known side effects that impact the entire body. The product must be put on/used properly. The manufacturer is exempt from product liability if the product is used improperly.

- The clothing sizes are to be observed.
- If your symptoms worsen or you notice any unusual changes, please contact your doctor immediately.

5 | Application

The incontinence pants are worn like ordinary underwear.

6 | Cleaning

The cleaning is done by usual household laundry, according to care instructions.

Note: Product may shrink slightly during drying.

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