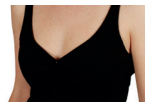
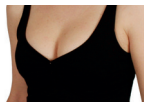
RUS

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
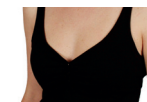
before

after

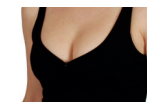With Silicone breastforms you can increase your cup size and enhance your figure without the risk or cost surgery.

Instructions for use

Directions Place them into your bra, to lift, support & enhance your breasts.

How To Use

Silicone breastforms are completely safe because they are used externally.

Silicone breastforms are so comfortable. They do not require adhesives that can pull, irritate and hurt the skin.

Silicone breastforms mold to fit the shape of your own breasts. You probably will forget you're wearing them.

The natural looking color is perfect to wear even under the lightest, sheerest clothing.

Silicone breastforms fill out your favorite clothes, from sweaters, to bathing suits.

Silicone breastforms can be worn at the beach even in salt water.

Silicone breastforms are Specially designed to have the natural look, feel and softness of real breasts.

Silicone breastforms are designed to bounce and move with your own breast tissue.

above

above

Centre

Attention!

Lotions or tanning agents may damage the film of the bra inserts. Cleanse your skin thoroughly before each application.

Care & Cleaning

Place your Silicone Breast Forms back into its original storage tray when not in use. Clean your Silicone Breast Forms after each time you wear them. Hand wash with cold or warm water with mild soap and pat dry with a soft cloth. Keep away from sharp objects. It can easily puncture unit.

Never place in washing machine, dryer or hot water.

Caution!

Keep out of reach of children. If any part of the product is ingested induce vomiting and consult a physician immediately.

Guarantee: The guarantee lasts for 24 months from the date of purchase for manufacturers' error in material and quality. This guarantee is only valid if the instructions in the "instructions for use" have been followed. The guarantee is no longer valid if the device is damaged due to violent usage, misuse or improper use or service by unqualified persons.

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